


ARC EQUIPMENT and SUPPLIES
1411 Broadway Santa Monica, CA 90404
310-394-1551 FAX 310-394-4832

Changing a Retraction Valve

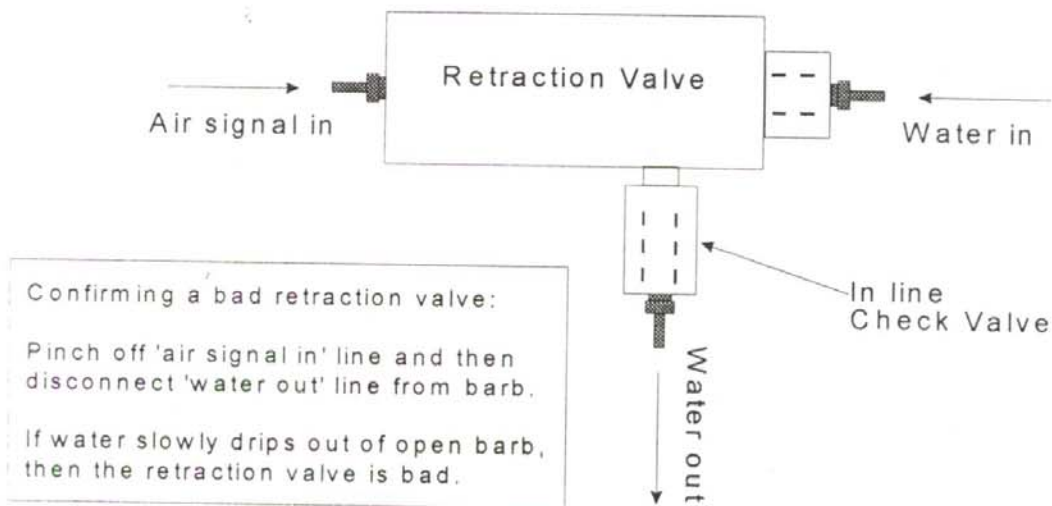

- 1) Pinch off 'water in' line going to retraction valve.
- 2) Carefully disconnect all three tubes connected to the retraction valve, noting which one goes to each location.
- 3) Unpinch 'water in' tubing while holding open end over sink or towel. Flush out water line for a few seconds, then pinch off 'water in' tube again.
- 4) Re-connect all three lines to the new retraction valve. Be sure each line goes back at the original position, as on the above diagram.

NOTE: The in line check valve threads into the retraction valve and can be re-used on the new retraction valve.